

# A-dec® Equipment Maintenance Checklist

## After Each Patient

---

- Clean, lubricate, and sterilize handpieces
- Clean syringe tips with ultrasonic and sterilize
- Wipe down and flush handpiece tubing
- For oral surgery procedures, flush vacuum line
- Clean/disinfect touch and transfer surfaces
- Replace all barriers

## Daily/As Needed

---

- Add A-dec ICX® to each water bottle refill<sup>†</sup>  
(see Table 3 for part numbers)
- Flush cuspidor for one minute
- Clean cuspidor bowl and bowl screen
- Use A-dec Evacuation Cleaner to flush vacuum lines<sup>†</sup> (see Table 3 for part numbers)

## Weekly/As Needed

---

Date

- Clean chair and stool upholstery with mild dishwashing liquid and water
- Clean HVE and saliva ejector valves
- Twice a week, inspect and maintain solids collector
- Maintain oil collector
- Lubricate handpiece chucks
- Confirm smooth handpiece bur rotation
- Clean dental light shield with mild dishwashing liquid and water
- If applicable, replace dental light Medicom® SafeShield™ barrier

---

---

---

---

---

---

---

---

## Monthly

---

Date

- Test dental unit water CFU level and shock waterlines with A-dec ICX Renew® or Restore®, as needed (see Table 3 for part numbers)
- Inspect all handpieces, motors, and O-rings; replace if worn/damaged
- Lubricate water bottle, HVE, and saliva ejector O-rings

---

---

---

## Yearly

---

Date

- Contact your local authorized A-dec dealer for installation of the annual preventative maintenance kit (see Table 1 for part numbers)
- Replace all handpiece and motor O-rings (see Table 2 for part numbers)

---

---

\*For detailed maintenance steps, refer to the *Instructions for Use* that came with your equipment or visit the A-dec Resource Center at [www.a-dec.com/resource-center](http://www.a-dec.com/resource-center).

<sup>†</sup>For detailed waterline maintenance instructions, refer to the *Waterline Maintenance Guide* (p/n 85.0983.00) that came with your equipment or visit the A-dec Resource Center at [www.a-dec.com/resource-center](http://www.a-dec.com/resource-center).

A-dec dental equipment is engineered to last. Regularly scheduled checkups and replacement parts ensure performance you can count on.

**Table 1: A-dec Maintenance Parts and Kits**

Item	Part Number
Silicone lubricant (5.3 oz tube)	98.1120.00
Screen trap (A-dec 300/500), 10 pack	11.1284.01
Syringe tips, 5 pack	23.0872.01
A-dec 500/300 Annual Preventative Maintenance Kit	90.1835.00
Performer®/Cascade®/200 Annual Preventative Maintenance Kit	90.1836.00
A-dec 500B 5-year Preventative Maintenance Kit	90.1870.00
A-dec 500A Surf (V2) 5-year Preventative Maintenance Kit	90.1874.00
A-dec 500A Surf (V1) 5-year Preventative Maintenance Kit	90.1873.00
A-dec 500A White/A-dec 300 (V2) 5-year Preventative Maintenance Kit	90.1872.00
A-dec 500A White/A-dec 300 (v1) 5-year Preventative Maintenance Kit	90.1871.00
Cascade/Decade®/Performer 5-year Preventative Maintenance Kit	90.1875.00
A-dec 200/Performer B (V1) 5-year Preventative Maintenance Kit	90.1876.00
A-dec 200/Performer B (V2) 5-year Preventative Maintenance Kit	90.1877.00

**Table 2: A-dec 501 Electric Motor O-rings**

Item	Part Number
O-ring, Motor Insert, Blue	54.0803.00
O-ring, Motor Insert, Black	54.0804.00
O-ring, Motor Tubing Connection	54.0805.00

**Table 3: A-dec 360 Products Recommended for Maintaining Dental Unit Waterlines (DUWL) and Evacuation Systems**

Use	Item	Part Number
Evacuation System	Evacuation System Cleaner, 33 oz bottle	91.0060.01
Evacuation System	Evacuation System Cleaner, 128 oz bottle	91.0061.01
DUWL Daily Maintenance	ICX Waterline Tablets, 0.7L (50 count)	91.1064.01
DUWL Daily Maintenance	ICX Waterline Tablets, 2L (50 count)	90.1065.01
DUWL Shock Treatment	ICX Renew, Box of 9 applications	90.1691.01
DUWL Shock Treatment	ICX Restore, Box of 9 applications (Canada only)	90.1821.01

Contact an A-dec authorized dealer for more accessory and replacement options.

