

## Specifications and Warranty


Warranty information is provided in the *Regulatory Information, Specifications, and Warranty document* (p/n 86.0221.00), which is available in the Document Library at [www.a-dec.com](http://www.a-dec.com).

A-dec Headquarters  
2601 Crestview Drive  
Newberg, OR 97132 USA  
Tel: 1.800.547.1883 Within USA/Canada  
Tel: 1.503.538.7478 Outside USA/Canada  
Fax: 1.503.538.0276  
[www.a-dec.com](http://www.a-dec.com)

A-dec Inc. makes no warranty of any kind with regard to the content in this document including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose.



86.0559.00 Rev B  
Copyright 2015 A-dec Inc.  
All rights reserved.

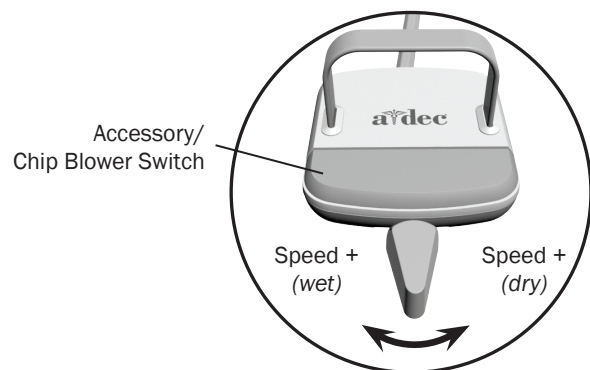

**NOTE** The lever foot control function can be reversed by a technician (change left direction to dry, right direction to wet). For more details, contact your authorized A-dec dealer.



Instructions for Use

## A-dec Lever Foot Control

### Overview

The foot control regulates drive air to the active handpiece and it provides an air signal that activates air coolant and water coolant flow.

The accessory/chip blower switch activates an accessory function or the chip blower. The chip blower sends a jet of air through the handpiece without activating the handpiece bur. The accessory function depends on your integrated accessory device. For more details, contact your authorized A-dec dealer.



### Handpiece Activation and Operation

To activate a handpiece, lift it from the holder or pull the whip forward. Use the foot control to perform the desired handpiece operation.

Operation	Procedure
Run a handpiece with water coolant.	Move the lever to the left. Move farther to increase speed.
Run a handpiece without water coolant.	Move the lever to the right. Move farther to increase speed.
Run the optional accessory or chip blower.	Press the accessory/chip blower switch.
Operate the intraoral camera.	Move the lever to the left or right to capture an image.



**NOTE** For handpiece adjustment information, refer to the instructions for use that came with your delivery system.