#### PRODUCT NAME: ICX

A-dec ICX tablets are specially formulated to maintain dental unit waterlines and prevent accumulation of odor and foul taste bacteria. Maintains dental unit waterline effluent ≤10 CFU/ml.

#### KEEP OUT OF REACH OF CHILDREN

#### WARNING

#### PRECAUTIONARY STATEMENTS

##### Hazards to Humans and Domestic Animals

Causes moderate eye irritation, wear protective eyewear. Avoid contact with eyes, skin or clothing. Wash hands thoroughly with soap and water after handling and before eating, drinking, chewing gum, or using tobacco.

#### DIRECTIONS FOR USE:

It is a violation of federal law to use this product in a manner inconsistent with its labeling.

ICX is intended for use in general dental applications by providing water to the dental syringe, various handpiece ancillary devices, and cuspidor cupfill. When used according to these instructions for every bottle refill, optimum efficacy for control of odor and foul taste bacteria can be achieved.

Before Refilling Bottle

- Empty the bottle of all remaining water.
- Drop tablet into an empty dental unit water bottle (0.7L tablet in 0.7 liter bottle, 2L tablet in 2 liter bottle). Avoid touching the tablet with unprotected skin.
- Fill bottle with water and connect to the dental unit.
- Wait 120 seconds for the tablet to fully dissolve.

**NOTE:** This product is intended for use with potable water and in conjunction with regular dental unit water testing, following flush (shock) treatment with an antimicrobial product registered with the U.S. EPA and recommended by the manufacturer.

#### STORAGE AND DISPOSAL

Do not contaminate water, food, or feed by storage or disposal. Store at room temperature out of direct sunlight; **DO NOT** store above 30°C (86°F). Nonrefillable container. Do not reuse or refill this container. Offer for recycling if available or dispose of empty outer foil pouch in the trash.

#### FIRST AID:

If solid product gets in eyes, hold eye open and rinse slowly and gently with water for 15 to 20 minutes. Remove contact lenses, if present, after the first five minutes, then continue rinsing.

#### CONTACT INFORMATION

If you have a question that is not addressed in this document, please contact A-dec Customer Service at one of the following phone numbers:

- 1.800.547.1883 (within USA and Canada)
- +1.503.538.7478 (outside USA and Canada)

Customer service is available Monday through Friday from 5 a.m. to 5 p.m. Pacific Standard Time (PST).

Report all serious incidents involving A-dec products to A-dec, Inc. If the incident occurs in the EU, also report it to A-dec’s EU Authorized Representative and the competent authority of the EU Member State in which the user/patient is established.

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